


# Aqua Splint Kit



## advantages compared to the classical rigid and soft splints:

- **immediate help** without impressions, bite registration or laboratory procedures
- **does not need grinding** or frequent recall visits (thanks to the hydrostatic balance between both water pads)
- **universal** and applicable in only a few minutes
- **replaces** both relaxation and distraction splints
- **great accuracy** and a excellent fit thanks to individual relining
- **no more than 10 hours** of daily wearing (8 hours by night, 2 hours by day, for 4 - 6 weeks)
- **excellent clinical results** and patient satisfaction thanks to rapid pain reduction, comfort and the advantageous positioning in the lower jaw (less visibility and more stability in comparison to the upper jaw)
- long life (approx. 4-6 weeks) and almost no fluid loss
- can be applied during **orthodontic treatment**

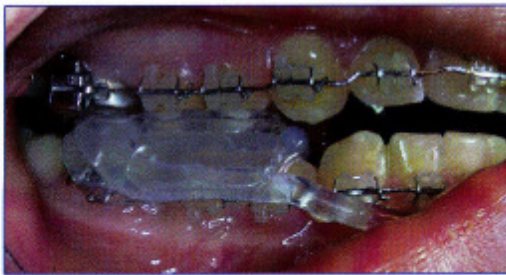

## Aqua Splint advantages compared to the traditional Aqua-Pads:

- Conventional Aqua Pads have to be held in position by permanent closure of the mouth and muscular activity, by what sufficient muscle relaxation can be prevented. The appropriate seat of AQUA SPLINT (relining) and the advantageous positioning in the lower jaw enable better muscle relaxation.




- The Aqua splint is the only self-adjusting customizable TMD splint that can also be used during an orthodontic treatment (block out brackets with protection wax beforehand)
- Long life (4-6 weeks) and almost no fluid loss
- No sharp edges



## Indication:


- **urgent pain management** in acute cases (e.g. disc displacement without reduction)
- **muscular relaxation**, particularly in case of acute or periodic grinding / pressing (bruxism)
- optimal initial TMD therapy
- **high differential diagnostic** informative value and supportive therapy in associated puzzling holistic diseases, e.g. cervical syndrome, tinnitus, migraine headache, stress syndrome, etc. (TMD is a multifactorial disorder. Malocclusion is not necessarily the main factor.)
- elimination of hidden **functional disturbances, primary contacts and forced bite**
- easy bite determination for the prosthetic / orthodontic rehabilitation or for the **permanent dental splint**
- support for **osteopathy / physiotherapy** following trauma, operations, limitations and blockades

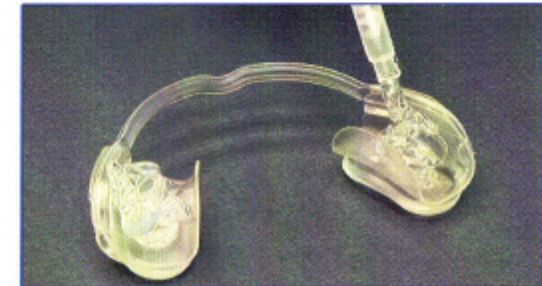

**AQUA SPLINT is a quality product MADE IN GERMANY and patented internationally.**

**WWW.AQUA-SPLINT.COM**

## Instructions

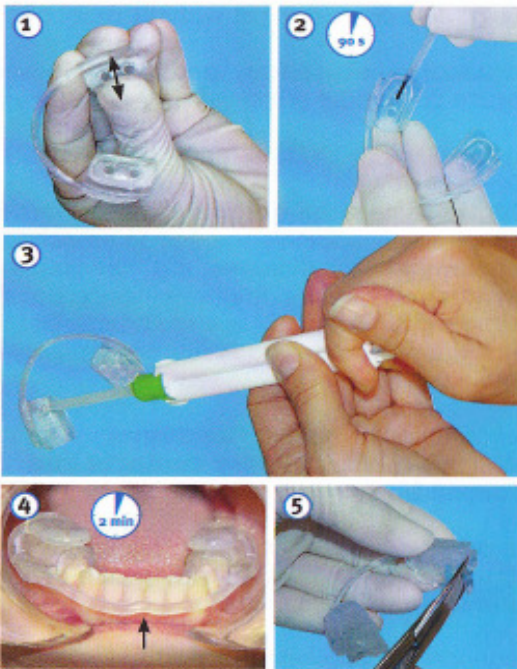

As easy as it looks

Further information like:

- seminar dates
- video demonstrations
- possibilities and limits of the Aqua Splint therapy

via [www.aqua-splint.com](http://www.aqua-splint.com)

Ordering via Phone / Fax:

+49 (0)371 433 02 09 +49 (0)371 402 03 59

Online-Shop: [www.TELEDENTA.com](http://www.TELEDENTA.com)

### Aqua Splint Kit

inkl. silicon relining, bonding und hygiene box


Product No.	Product	Price	Pieces
400-001-01	Aqua Splint Kit	39,90 €	
400-001-02	Aqua Splint Kit - Pack of 3	110,00 €	
400-001-04	Aqua Splint Kit - Pack of 5	179,50 €	

### TMD Aqua Splint Concept (DVD)

This DVD offers an optimal addition for the correspondent course, for patient consulting and the education of colleagues.

#### Content:

- fast initial diagnosis of the TMD with the help of manual diagnostics
- simplified differential diagnostic of the TMD with the help of Aqua Splint
- application and management of the Aqua Splint
- manual / physiotherapy exercises as accompanying therapy
- occlusal rehabilitation after the Aqua Splint

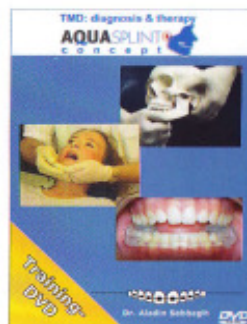

Product No	Product	Price	Pieces
400-002-02	TMD Aqua Splint Concept (DVD)	79,90 €	

Adress / Stamp



TeleDenta GmbH  
Hainstr. 108  
09130 Chemnitz  
Germany

Telefon: +49 (0) 371 4330209  
Fax: +49 (0) 371 4020359  
E-Mail: [info@teledenta.com](mailto:info@teledenta.com)  
Web: [www.teledenta.com](http://www.teledenta.com)

All offerings are directed solely at businesses.  
All prices plus postage and tax. Valid until 2011/10/31.  
No responsibility for errors or misprint.

The self-adjusting  
adaptable CMD Splint

THE WORLD'S FIRST!



- **immediate help**, Pain relief and muscle relaxation
- **without** impressions, bite registration, grinding or laboratory procedures
- **great fitting accuracy** through individual silicon relining
- **replaces** both relaxation and distraction splints
- enables the joint to **naturally balance itself** (Pascal's Law)
- **differential diagnostic** for associated holistic diseases

for easy and effective  
TMD  
diagnosis & therapy

[WWW.AQUA-SPLINT.COM](http://WWW.AQUA-SPLINT.COM)